

Oral & Nasal Suction Device & Cover

Indications for Use

BoogieBaby offers a non-invasive suction alternative for the nasal cavity for use with newborns, infants and children. Indicated for use as a vacuum-powered suction device to aspirate fluids from the mouth and nares.

Instructions For Use

Single Patient Use.

For oral and nasal use only.

Packaged clean & ready to use.

1. Connect to standard suction line.
2. Set suction pressure and use per hospital protocol.
3. Suction sterile water through device after each use.
4. Replace at least every 24 hours.

Product Specifications

Latex-Free, DEHP-Free, BPA-Free. Packaged in individual polyethylene bags. Designed & engineered by SandBox Medical in the USA. Manufactured in China at an FDA registered & inspected facility.

WARNINGS

Inserting the tip too far into the mouth or nares can cause tissue trauma. The suction device & cover should be kept free of debris. Federal Law (USA) restricts sale of this device to, or on the order of a physician.